3DELTA CROWN

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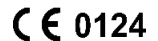
3DELTA CROWN

Description

3DELTA CROWN is a flowable, light-curing, acrylate-based composite material for the additive production of anterior and posterior restorations and dentures. **3DELTA CROWN** is optimized for wavelengths of 385 nm in layer thicknesses of 100 µm and can be used with 3D printers of following manufacturers:

- Rapidshape [D-Series (D20II, D30II, D40II, D10+, D20+ cartridge, D20+, D30+, D50+), PRO-Series (PRO 20)]
- Asiga [PRO4K UV / MAX UV]

The materials satisfy the performance requirements of DIN EN ISO 10477 type 2/ class 2 and DIN EN ISO 20795-1 type 4

When using **3DELTA CROWN**, the user must follow all the relevant instructions, including the instructions for use, installation directions, operating instructions, user manuals and other related documents for all listed compatible devices. Strict adherence to all the instructions is crucial for guaranteeing a reliable printed result. Ensure that the equipment items are installed, maintained and calibrated in accordance with the manufacturer's operating instructions. Use only the above-mentioned validated and authorized printers.

Composition

Preparation consisting of acrylates, fillers (50 %/wt., Ø 0.7 µm), initiators and pigments.

Intended Purpose

Replacement of tooth substance.

Indication

Additive production of anterior and posterior restorations, e.g.

- Single-tooth crowns,
- Veneers, inlays, onlays,
- Artificial denture teeth
- Temporary bridges
- Monolithic dentures

Contraindications

Patient and user: Products made from **3DELTA CROWN** should not be used in patients who are known to be allergic to any of the ingredients. No other patient restrictions apply.

Adverse reactions

Patient and user: Product can cause allergic reactions.
User: Observe warnings.

Interactions

Cements containing eugenol must not be used.

Processing instructions

DESIGN GUIDELINES FOR CROWNS AND BRIDGES

- ↳ Min. wall thickness occlusally 1.5 mm in the central fissure.
- ↳ Min. circumferential wall thickness 1 mm.
- ↳ Min. connector cross-sectional area 12 mm² anterior / 14 mm² posterior.

CONSTRUCTION PROCESS

↳ **Prepare data set:** The orientation and type of support structures can be found in the system supplier's instructions for use.

↳ **Establish variable process parameters:** The appropriate process parameters are selected according to the specifications of the printer system supplier. Ensure that the correct material-specific set of parameters is selected (product, layer thickness, exposure time).

↳ **Fill with material:** Thoroughly stir material with a spatula and fill the reservoir of the device with the material. Then wait 10 min.

↳ Start construction process.

CLEANING AND DRYING

↳ **Option 1:** The component is cleaned by removing the adherent resin using a centrifuge. The recommended rotational speed and time is 1500 rpm for 2 minutes.

Recommendation 1: Laboratory centrifuge with tube holder, e.g.

- SBS-LZ-4000/20-6 from Steinberg
- XC 2450, from Premiere

→ Wrap objects in cellulose cloth, place centrally in the tube and centrifuge, then turn the tube through 180° and centrifuge again.

Recommendation 2: Plate centrifuge e.g.

- LMC 3000 from Biosan with R-2 swing-out rotor.

→ Wrap objects in cellulose cloth, secure centrally on the plate holder of the rotor and centrifuge, then turn the tube through 180° and centrifuge again.

↳ **Option 2:** The component is cleaned under flowing isopropanol for **max. 5 min.** If necessary, carefully clean the objects with a brush and a spray bottle filled with isopropanol.

↳ **Dry component:** Blow component with compressed air and leave to dry for 30 min at room temperature or in a drying cabinet adjusted to a temperature of approx. 37 °C (99 °F).

POST CURING

For post-curing, the components must be dry and cooled to room temperature.

Used printer	Curing Device	Post-Curing
Rapidshape, Asiga	Otoflash (NK Optik)	2000 flashes each for top and bottom ¹
	CURE (Rapidshape)	Selection of the corresponding material parameter set ²

¹ Recommendation: Curing under exclusion of oxygen prevents the formation of the inhibition layer.

² Exposure time: 8 min / vacuum 50 mbar / without nitrogen / heater off / power 100 % / wave length 375+415 nm.

Ensure that the unit used is serviced and calibrated in accordance with the manufacturer's instructions in order to preserve its mechanical properties, color and biocompatibility.

FINAL CLEANING

↳ **Final clean component:** The component is cleaned under flowing isopropanol for **max. 2 min.** If necessary, carefully clean the objects with a brush and a spray bottle filled with isopropanol.

↳ **Dry component:** Blow component with compressed air and leave to dry for 30 min. Alternatively, the objects can be stored in a drying cabinet adjusted to a temperature of approx. 37 °C (99 °F).

REMOVE SUPPORT STRUCTURE AND FINISH

↳ Remove supports and grind remains of the supports to a smooth surface.

↳ Sandblast inner and outer surfaces of the part with 50 µm corundum at 1.5 bar to remove white layer.

↳ **Customization, characterization, polishing:** Restorations made from **3DELTA CROWN** can be supplemented and customized with light-curing composites and stains. In such cases, the processing instructions of the respective materials used must be observed. Reductions in the

vestibular or incisal area should not exceed 0.3 mm. Restorations made from **3DELTA CROWN** can be polished to a high gloss finish according to the usual dental techniques.

FINISHING COMPLETED

Further processing instructions:

The fit of the printed restorations must be checked and assessed by the dental technician (on the model) and dentist (in the patient's mouth). Before placing it in the patient's mouth, the restoration can be cleaned with alcohol.

Fixation

Restorations can be fixed with self-adhesive cements (e.g. RelyX™ Unicem 2 / 3M ESPE) or resin cements (e.g. Variolink Esthetic DC / Ivoclar Vivadent) with primers (e.g. Adhese Universal / Ivoclar Vivadent). Eugenol-free temporary luting cements (e.g., Temp-Bond / Kerr) are used for temporary cementation. Printed denture teeth can be bonded to the denture base with **3DELTA DENTURE BOND** and **3DELTA DENTURE FIX** (DeltaMed GmbH). Observe the processing protocol of the materials used, particularly including the protocol for the pre-treatment of the restoration.

Warnings

Any deviation from the processing process described here can jeopardize the biocompatibility, functionality and safety of the final product. Please always observe the respective valid safety data sheets. Contains skin sensitizing substances: acrylates, phosphine oxide. Applies to the unpolymerized material: May cause an allergic skin reaction. Avoid contact with unpolymerized material. Wear protective gloves / protective clothing / eye protection. If on skin: Wash with plenty of soap and water. If skin irritation or rash occurs: Get medical advice / attention. Avoid breathing dust / vapours / spray. Switch on extraction system. The unpolymerized material is harmful to the environment. Avoid release to the environment. Collect spillage. Always observe the relevant country-specific health and safety regulations and legal requirements when handling hazardous substances.

Storage instructions / shelf life

Protect from exposure to light. May polymerize spontaneously. Store between 4 °C and 25 °C (39.2 °F and 77 °F). Keep containers tightly closed. Do not use after the expiration date. See imprint/label for expiration date.

Disposal instructions

Disposal of contents/container in accordance with local regulations.

Delivery Units

		REF 500 g	REF 1000 g
3DELTA CROWN	A1	1611110	1611210
3DELTA CROWN	A2	1611120	1611220
3DELTA CROWN	A3	1611130	1611230
3DELTA CROWN	A3.5	1611135	1611235
3DELTA CROWN	B1	1611155	1611255
3DELTA CROWN	B3	1611165	1611265
3DELTA CROWN	C1	1611160	1611260
3DELTA CROWN	C3	1611170	1611270
3DELTA CROWN	D3	1611175	1611275
3DELTA CROWN	Bleach	1611150	1611250

The product was developed for use in dentistry and must be used according to the instructions for use. Serious incidents must be reported to DeltaMed GmbH and the responsible authority. For the Summary of Safety and Clinical Performance (SSCP) please consult the Eudamed database (<https://ec.europa.eu/tools/eudamed>) or contact sscp@deltamed.de.

The most recent instructions for use can be found at deltamed.ifu.dental. A printed version is available from the manufacturer upon request and will be provided to the user free of charge within seven days.

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