# **ceramill**<sup>®</sup> liquid fx

<b>DE</b> Gebrauchsanweisung	3 - 16
<b>EN</b> Instruction Manual	17 - 30
<b>FR</b> Instructions d'utilisation	31 - 44
<b>IT</b> Istruzioni per l'uso	45 - 58
<b>ES</b> Instrucciones de uso	59 - 72

- Original Gebrauchsanweisung -

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## Table of Contents

<b>Explanation of Symbols</b> .....	<b>18</b>
<b>General Safety Instructions</b> .....	<b>20</b>
<b>Suitable Personnel</b> .....	<b>21</b>
<b>Properties</b> .....	<b>22</b>
<b>Warranty/Exclusion of Liability</b> .....	<b>23</b>
<b>Application</b> .....	<b>24</b>
<b>Pre-drying and Sintering</b> .....	<b>27</b>
<b>Environmental Protection</b> .....	<b>30</b>
<b>Accessories</b> .....	<b>30</b>
<b>Download Infos</b> .....	<b>30</b>

## Explanation of Symbols

### Warning indications

Warning indications in the text are marked with a triangle and boxed.

In case of hazards through electricity, the exclamation mark in the warning triangle is substituted by a lightning bolt.

Signal words at the beginning of a warning indication specify the type and severity of the consequences, if the measures to avert the hazard are not adhered to.

- \_ **NOTE** means that property damage can occur.
- \_ **CAUTION** means that light to fairly serious personal injury can occur.
- \_ **WARNING** means that serious personal injury can occur.
- \_ **DANGER** means that life-threatening personal injury can occur.

### Important information

Important information that do not lead to hazards for humans or to property damage are marked with the icon aside and are boxed.

### Other symbols in the Manual

Symbol	Meaning
▷	Item of an operation description
—	Item of a list
•	Subitem of an operation description or a list
[3]	Numbers in square brackets refer to position numbers in graphics/figures

### Other symbols on the product

Symbol	Meaning
	Article number
	Batch code
	Manufacturer
	Observe instructions for use
	Expiration date
	Caution
	Caustic
<b>Rx only</b>	According to U.S. Federal Law, the product may only be sold through or on behalf of a dentist.

## General Safety Instructions

EN

**CAUTION:**

- ▷ If medical advice is needed, have product container or label at hand.
- Keep out of reach of children.
- ▷ Read label before use.
- ▷ Do not breathe dust/fume/gas/mist/vapours/spray.
- ▷ Wear protective gloves/protective clothing/eye protection/face protection.
- ▷ IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.
- ▷ Immediately call a POISON CENTER/doctor.
- ▷ Store locked up.
- Dispose of the contents/container in accordance with local/ regional/ international regulations.

For further information, such as e.g. the safety data sheets on Ceramill Liquid, please refer to [www.amanngirrbach.com](http://www.amanngirrbach.com) in the download area.

**NOTE:**

Damage to the environment!

The product contains heavy metals. Prevent emission into the environment. Specific pre-treatment is required.

Water-pollution class 1 (self-classification): slightly dangerous to water.

▷ Do not allow undiluted product or large quantities of it to be introduced into groundwater, bodies of water or the sewage system.

**NOTE:**

The Ceramill zirconium oxide blanks *Preshade/Multilayer* are not released for application with Ceramill Liquid.

- ▷ Use Ceramill Liquid only with white Ceramill zirconium oxide.
- ▷ Do not mix Ceramill Liquid with other Ceramill Liquid systems. The staining solutions may only be combined unmixed within a Ceramill Liquid system.

## Suitable Personnel

**NOTE:**

The product may only be used by trained dental technicians.

## Properties

### Intended use

Dye solutions for staining zirconium oxide blanks.

### Product description

Ceramill Liquid FX are water-based staining solutions for complete or partial staining of restorations of white Ceramill Zolid FX prior to the sintering process.

Ceramill Liquid FX is available in the 16 basic shades (dentine colours), as well as in various effect shades for individualisation.

### Safety data sheet/Declaration of conformity

The data sheet on safety and the declaration of conformity are available on request under [www.amangirrbach.com](http://www.amangirrbach.com).

### Shelf life

Ceramill Liquids FX have a shelf life of 2 years.

The expiry date and batch number are indicated on the original packaging.

## Storage

- ▷ Store in original container only.
- ▷ Store separated from foodstuffs.
- ▷ Keep container tightly sealed.

Within their shelf life, the staining solutions can change their natural colour during storage or usage. The colouration of the sintered restoration is not affected through this.

## Warranty/Exclusion of Liability

Application-technical recommendations, whether given orally, in writing or in the course of practical training, are guidelines. Our products are subject to continuous further development. Subject to changes in handling and composition.

## Application

The shade allocations of the dye solutions were produced based on sample restorations with a standardised frame thickness **GS** of 1.2 mm. For this frame thickness GS, the Ceramill Liquid FX 5 x were applied on the inside and outside of the crowns with a size 5 brush.

For restorations with a thinner or thicker frame thickness GS, the Ceramill Liquid FX must be applied accordingly less or more often. Due to the high translucency of Ceramill Zolid FX, it is recommended to apply the Ceramill Liquid FX effect shades less intensively.

The shade allocations are to be considered as suggestion on behalf of Amann Girrbach. The shade perception is subjective and can differ considerably from user to user. Before staining the objects, it is recommended to create colour samples.

## Colouring of frames through immersion

### NOTE:

Before staining:

- ▷ Clean the frames from residual zirconia dust using a brush and compressed air.

**NOTE:**

Ensure there are no metal traces on the objects.

- ▷ Use only plastic tweezers for immersing.

Process the milled frames of Ceramill Zolid FX as follows:

- ▷ Select the desired staining solution according to colour coding A-D or according to shade-assignment table.
- ▷ Depending on processing technique, thin the selected staining solution with Ceramill Dimmer Liquid to achieve the desired dentine shade.
- ▷ Immerse the frame into the staining solution according to the processing technique.
- ▷ Dab off excess liquid with a lint-free disposable cloth.
- ▷ Recommendation: Pre-dry for 1 h (see page 13)
- ▷ Sinter in a sintering bowl in the Ceramill sintering furnace.

For further information and details on the immersing technique, such as shade-assignment tables, mixing information or immersing durations, please refer to the illustrated instructions on staining techniques under

*[www.amanngirrbach.com](http://www.amanngirrbach.com).*

## Staining of monolithic crowns and bridges with brush technique

### NOTE:

Before staining:

- ▷ Clean the frames from residual zirconia dust using a brush and compressed air.

### NOTE:

Ensure there are no metal traces on the objects.

Metal ferrules on the brush can leave traces on the zirconium oxide and additionally influence the staining solution!

- ▷ Use only metal-free brushes with synthetic bristles.

Brushes with synthetic bristles are less sensitive against staining solutions and alcohol than brushes with real hair.

- ▷ As soon as a new mixing ratio or a different shade is being used, wash out reusable brushes with alcohol (e.g. ethanol) and afterwards with distilled water.

For further information and details on the brush technique, such as shade-assignment tables and mixing information, please refer to the illustrated instructions on staining techniques under [www.amanngirrbach.com](http://www.amanngirrbach.com).

## Pre-drying and Sintering

### Pre-drying

After staining with Ceramill Liquid FX, the Ceramill Zolid FX should be pre-dried. This is to avoid the formation of spots (homogenisation). The risk of cracks and fissures is reduced.

Material	Pre-drying temperature	Pre-drying duration
Ceramill Zolid FX frames	80 °C	60 min

### Suitable equipment

After staining the Ceramill Zolid FX, the work can be dried using a ceramic furnace. The ceramic furnace and the further ceramic firing cycles are not influenced through the drying procedure. However, it is recommended to occasionally burn out the ceramic furnace at 1150 °C for 5 minutes. As an alternative, a drying cabinet or mini oven can also be used.

Suitable equipment for pre-drying:

- \_ Domestic ovens/mini ovens ✓✓✓
- \_ Drying cabinet ✓✓✓
- \_ Ceramic furnace ✓✓
  - In case the ceramic furnace cannot be programmed to 80 °C, enter the lowest possible temperature and leave furnace door open.
  - Use a thermometer to check, whether the 80 °C are reached on the object.
- \_ Ceramill Therm (from version 2008 on) ✓
- \_ Curing lamps ✓
  - Curing lamps generally do not achieve the recommended drying temperature. Therefore, when using curing lamps, the drying duration must be accordingly prolonged.

## Sintering

### Basic rule on sintering

All stained restorations **without** auxiliary sintering block (to 3 units max.) can principally be sintered in standard program 1.

## Adjustments on the sintering furnace

▷ Adjust the following parameters on the sintering furnace:

Material	Sintering furnace	Sintering program	Recommended pre-drying
Ceramill Zolid FX	Therm 1	Program 1	1 h at 80 °C (external, e.g. ceramic furnace)
	From Therm 2 on	Program 1	1 h at 80 °C (external, e.g. ceramic furnace or programmable on program position 4)

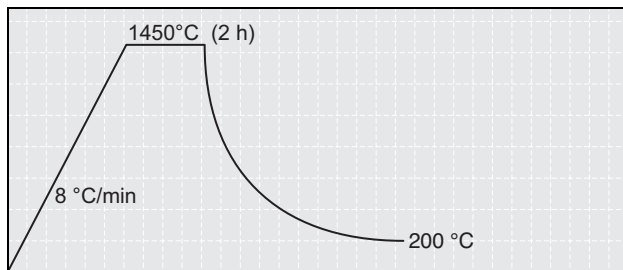
Fig. 1 Standard sintering program Ceramill Zolid FX in the Ceramill Therm 2

▷ Sinter in a sintering bowl in the Ceramill Therm.

## Environmental Protection

### Packaging

All packaging materials used are environmentally-friendly and recyclable.

### Disposal

- \_ Ceramill Liquid may not be disposed of together with household waste.
- \_ Do not dispose of into sewer systems.
- \_ Dispose of the contents/container in accordance with local/ regional/ international regulations.

## Accessories

For more information on product-specific accessories, please refer to [www.amanngirrbach.com](http://www.amanngirrbach.com).

## Download Infos

Further instructions/manuals, safety data sheets and video tutorials can be downloaded under [www.amanngirrbach.com](http://www.amanngirrbach.com).

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**Amann Girrbach AG**
**Amann Girrbach GmbH**