Gebrauchsanweisung  
Instructions for use  
Mode d'emploi  
Istruzioni per l'uso  
Instrucciones de empleo

Manufacturer | Hersteller  
Distribution | Vertrieb

Distribution | Vertrieb D/A

**Amann Girrbaach AG**  
Herrschaftswiesen 1  
6842 Koblach | Austria  
Fon +43 5523 62333-105  
Fax +43 5523 62333-5119  
austria@amanngirrbaach.com

**Amann Girrbaach GmbH**  
Dürrenweg 40  
75177 Pforzheim | Germany  
Fon +49 7231 957-100  
Fax +49 7231 957-159  
germany@amanngirrbaach.com  
[www.amanngirrbaach.com](http://www.amanngirrbaach.com)

Made in the European Union

ISO 9001

www.heilblau.com 32063-FB - 2015-09-14

DE

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Original-Gebrauchsanweisung

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EN

## INSTRUCTIONS FOR USE

Translation of the original instructions for use

### PURPOSE

Wax blank for processing in a milling machine.

### PRODUCT DESCRIPTION

White milling wax for use in a milling machine for being cast, pressed or over-pressed.

### TECHNICAL DATA

Colour:	white to cream colours	
Dripping point:	100 - 130 °C	ASTM D 3954
Viscosity at 140 °C:	~ 650 mPa	DIN 53018
Solubility in water:	insoluble	
Acidity:	0 mg KOH/g	ISO 2114
Saponification number:	0 mg KOH/g	ISO 3681
Density at 23 °C:	0.92 - 0.96 g/cm <sup>3</sup>	DIN 53479
Odour:	odourless	

Milling wax can be burned out without residue.

### CHEMICAL COMPOSITION (PERCENTAGE BY WEIGHT)

100 %	Polyethylene wax
-------	------------------

### STORAGE

Store in a dry place in its original packaging.  
(< 25 °C). Keep away from direct sunlight.

### WARNING AND SAFETY INSTRUCTIONS

When processing, please use personal protective clothing (dust mask, protective goggles, ...).

### INDICATIONS

- \_ Can be cast, pressed or over-pressed.
- \_ No clinical dental product, not suitable for use in the oral cavity.

### DESIGN OF THE SUBSTRUCTURE

Follow the manufacturer's instructions for materials in which the substructure is cast or pressed.

### DISPOSAL

Dispose of in accordance with official regulations.

### GUARANTEE

Application recommendations, no matter whether verbally, in writing, or by means of practical instructions are based on our experience and tests and are thus only a recommendation. Our products are subject to continuous further development. We reserve the right to alter the products in terms of design or composition.

### SAFETY DATA SHEETS AND DECLARATION OF CONFORMITY

Safety data sheets and declaration of conformity are available on request and under [www.amanngirrbaach.com](http://www.amanngirrbaach.com)

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