


ceraMotion®

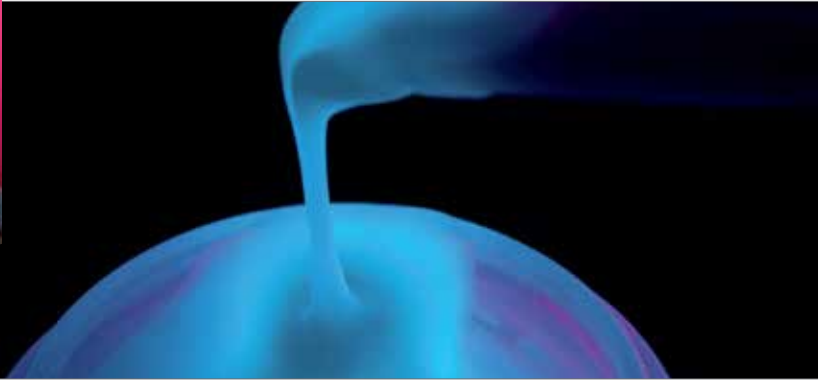

my first
ceramic
in paste



ceraMotion® Zr Paste Glaze.

Perfect glaze material for zirconium oxide and lithium disilicate.

Perfect glaze in **one** firing!

2 glaze pastes for the perfect glaze in one firing. Ready-to-use, fluorescent glaze material for zirconium oxide and lithium disilicate:

ceraMotion® Zr Paste Glaze transpa, 3g | REF 250-271-03

ceraMotion® Zr Paste Glaze bright, 3g | REF 250-272-03

Shared advantages:

- ✓ Only one glaze firing needed
- ✓ Ready-to-use paste
- ✓ Natural fluorescence
- ✓ Excellent wettability
- ✓ Homogeneous glaze



ceraMotion® Zr Paste Glaze transpa




ceraMotion® Zr Paste Glaze bright



The difference.

*medium
brightness value*

*higher
brightness value*

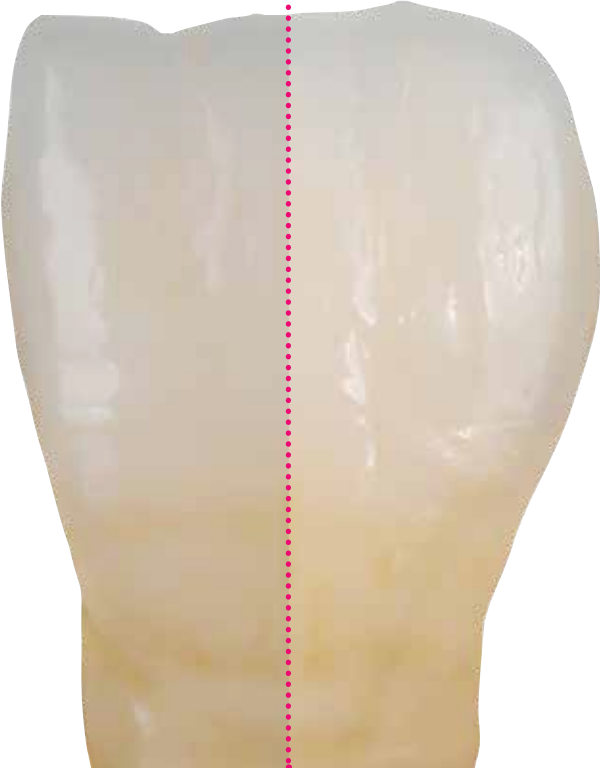

The fluorescence of a zirconium oxide crown....



...of a natural tooth.



*natural
fluorescence*



All ceraMotion® One Touch pastes are fluorescent and have been adapted to reflect the

fluorescence
of natural teeth.

ceraMotion® Zr Paste Glaze.



*"ceraMotion® Zr Paste Glaze
and ceraMotion® One Touch:
easy, quick and aesthetic"*

MDT Germano Rossi



More information 



Natural glaze with zirconium oxide and lithium disilicate.



Firing Chart.

	Start temp. (°C/°F)	Drying time (min.)	Heat rate (°C / °F per min.)	Vacuum start (°C/°F)	Vacuum end (°C/°F)	Firing temperature (°C/°F)	Holding time (min.)
Glaze firing	450	6	55	450	730	730	1 (without vacuum)

! The firing temperatures indicated are guidelines and may deviate depending on the furnace used.

Dentaurum

Germany | Benelux | España | France | Italia | Switzerland | Australia | Canada | USA
and in more than 130 countries worldwide.



DENTAURUM
QUALITY
WORLDWIDE
UNIQUE



www.dentaurum.com



Date of information: 07/20
Subject to modifications

D
DENTAURUM

Turnstr. 31 | 75228 Ispringen | Germany | Phone +49 72 31/803-0 | Fax +49 72 31/803-295
www.dentaurum.com | info@dentaurum.com