CE 0483

Herbst I

- Informationen zu Produkten finden Sie unter www.dentaaurum.com
- For more information on our products, please visit www.dentaaurum.com
- Vous trouverez toutes les informations sur nos produits sur www.dentaaurum.com
- Descubra nuestros productos en www.dentaaurum.com
- Informazioni su prodotti sono disponibili nel sito www.dentaaurum.com

Stand der Information | Date of information | Mise à jour |
Fecha de la información | Data dell'informazione: **05/18**

Änderungen vorbehalten | Subject to modifications | Sous réserve de modifications | Reservado el derecho de modificación |
Con riserva di apportare modifiche

Turnstr. 31 | 75228 Ispringen | Germany | Tel. +49 72 31 / 803-0 | Fax +49 72 31 / 803-295
www.dentaaurum.com | info@dentaaurum.com

ES

Estimado cliente

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EN

Dear Customer

Thank you for choosing a quality product from Dentaurum.

It is essential to read these instructions carefully and adhere to them to ensure safe, efficient use and ensure that you and your patients gain full benefit.

Instructions for use cannot describe every eventuality and possible application. In case of questions or ideas, please contact your local representative.

As our products are regularly upgraded, we recommend that you always carefully read the current instructions for use supplied with the product and stored in the internet at www.dentaurum.com, even though you frequently use the same product.

1. Manufacturer

Dentaurum GmbH & Co. KG
Turnstr. 31 | 75228 Ispringen | Germany

2. Fabricating the Herbst I appliance

Pour the impressions, which include the waxed out bands fitted on the 1st premolars and molars, with dental stone (e. g. Rapidur®). Mount the models in the correct position in an articulator with plaster using the activating bite recorded by the operator. Bend and laser weld the half-round remaloy® wire (ø 0.75 x 1.5 mm) or round remaloy® wire (ø 1.0 mm) to the bands to form a lingual arch. A palatal bar can be placed in the upper jaw between the first molars to provide stabilization.

Next spot weld and solder the bases of the hinge units to the bands (e. g. using the spot welding unit Master 3000 or laser). Ensure that the screws are at the correct angle to the axis between the upper and lower jaw. It is also important that the hinges between the upper and lower jaw operate without friction to ensure smooth functioning of the telescopic mechanism. Friction can damage the appliance – especially the bands.

Then fit the telescope tubes and telescope rods: first shorten the tubes according to the activating bite and then fit the telescope rods, which should be approx. 2 mm longer.

If the telescope rods are too short, they could come out of the telescope tube; if the rods are too long, they could cause irritation of the mucosa (risk of injury!). We therefore recommend that the exact length is always checked again intraorally.

Note: “Double bands” should be used to minimize the risk of the bands being damaged by shear forces. This involves welding a second band over the band already fitted (e.g. using the spot welding unit Master 3000).

Generally the second band is 2 to 3 sizes larger than the first band fitted. Ready-welded “double bands” can be ordered from Dentaurum. The order form is available on request.

Fitting intraorally:

The bands are fitted intraorally using standard band cement. We recommend covering the bases of the hinges with wax to ensure that the cement does not penetrate into the thread. After the cement has set, attach the telescope tubes and telescope rods (already shortened!) to the bases.

If necessary, the Herbst I appliance can be activated using various spacers (see Product range).

3. Composition

Information about the composition is included in the materials list, see catalog or www.dentaurum.com.

4. Safety instructions

The fit of the Herbst I hinge and tightness of the screws should be checked at regular intervals to prevent the appliance spontaneously coming apart and the components being swallowed or aspirated! If necessary, tighten the screws and secure them with a special screw fixing adhesive, e.g. Ceka® Bond.

The product should not be used if there is a known allergy to one or more of the ingredients.

5. Information on single-use products

The Herbst I appliance is intended for single use only. Reconditioning of Herbst I that has already been used (recycling) and its reuse on a patient is not permitted.

6. Product range

Herbst I set	REF 607-120-00	1 set
with hexagon socket screws		
For welding or soldering to bands or cast splints.		
Contents:		
<ul style="list-style-type: none">1 x telescope tube, right 1 x telescope tube, left 2 x telescope rod 4 x hexagon socket screw 4 x F-base 1 x Instructions for use		
The hexagon socket screw key is not included in delivery.		
Hexagon socket screw key	REF 607-144-00	1 piece
for Herbst TS /I/II/SUS ² sterilizable		
wrench size 1.5 mm		
Allen screws	REF 607-122-30	10 pieces
for Herbst TS /I		
Herbst F base	REF 607-127-30	10 pieces
for Herbst TS /I		
Telescope tube		
for Herbst I /II,		
left	REF 607-121-00	1 piece
right	REF 607-122-00	1 piece
Telescope rod	REF 607-124-00	1 piece
for Herbst I /II		
Spacers		
for Herbst I /II /IV		
Length: 1 mm	REF 607-103-00	10 pieces
Length: 2 mm	REF 607-104-00	10 pieces
Length: 3 mm	REF 607-105-00	10 pieces
Length: 4 mm	REF 607-107-00	10 pieces
Length: 5 mm	REF 607-112-00	10 pieces

7. With reference to our quality standards

Dentaurum ensures a faultless quality of the products manufactured by us. These recommendations are based upon our own experiences. The user himself is responsible for the processing of the products. Responsibility for failures cannot be taken, as we have no influence on the processing on site.

8. Explanation of symbols used on the label

 Please refer to the label. Additional information can be found at www.dentaurum.com (Explanation of symbols REF 989-313-00).

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