



Thermo forming blanks

Dreve Dentamid GmbH • Max-Planck-Straße 31 • 59423 Unna/Germany

Tel +49 2303 8807-0 • Fax +49 2303 8807-55 • E-Mail info@dreve.de • www.dreve.com

All the Dreve materials have a regulatory diameter of 120 mm. All item numbers listed up on the info pages refer to this standard size.

The following extra sizes are available on request:

- blanks with a diameter of 125
- square-shaped blanks with a side length of 120 and 125 mm

The given heating periods:



- apply to pre-heated machines
- can change by the ageing process of the heating elements
- can shorten in accordance with the total warming of the unit

On the front side of the info pages you can find the material for the corresponding application; the item number and packing units are to be found on reverse of the info pages.

Standard size



Extra size

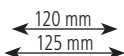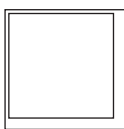

In general


Drufoplast 2 / 3 mm pink





- very break-resistant
- suitable tray handles available

Drufoplast (includes tray handle) 3.0 mm clear-transparent

- very break-resistant

Warm-up times  in minutes

Cooling times  in minutes and under pressure res. vacuum

	Drufosmart	Vacformat U	Vacformat 2000	
				
Drufoplast 2 mm	1:35	1:30	1:05	1:30
3 mm	1:55	1:55	1:20	2:00
Drufoplast (incl. tray handle) 3 mm	2:05	2:10	1:35	2:10

Barcode information for Drufomat-scan and Drufosmart-scan please find on page „Barcodes“

Drufoplast 2 mm

D-4210Z pink 15 pieces

Drufoplast 3 mm

D-42081Z pink 30 pieces

Drufoplast (includes tray handle) 3.0 mm

D-4214 transparent 10 pieces


D-4215 transparent 100 pieces





Drufoplast 3 mm pink

- very break-resistant
- suitable tray handles available

Drufoplast (includes tray handle) 3 mm clear-transparent very break-resistant

Warm-up times  in minutes

Cooling times  in minutes and under pressure res. vacuum

	Drufosmart	Vacfomat U	Vacfomat 2000	
				
Drufoplast 3 mm	1:55	1:55	1:20	2:00
Drufoplast (incl. tray handle) 3 mm	2:05	2:10	1:35	2:10

Barcode information for Drufomat-scan and Drufosmart-scan please find on page „Barcodes“

Drufoplast 3 mm

D-42081Z pink 30 pieces

Drufoplast (includes tray handle) 3 mm

D-4214 transparent 10 pieces

D-4215 transparent 100 pieces


Individual trays





Biolit D 1 / 2 mm

clear-transparent

- material can be sterilised
- predrying required
(4 hours at 80 °C per mm blank thickness)
- clear-transparent
- bonds with acrylate

Warm-up times  in minutes

Cooling times  in minutes and under pressure res. vacuum

	Drufosmart	Vacfomat U	Vacfomat 2000	
				
Biolit D				
1 mm	1:45	1:45	1:35	1:45
2 mm	2:15	2:15	2:05	2:15

Barcode information for Drufomat-scan and Drufosmart-scan please find on page „Barcodes“

Biolit D 1 mm

D-4260 clear-transparent 10 pieces

Biolit D 2 mm

D-4262 clear-transparent 10 pieces

Bandage plates

Biolon 0.5 / 0.75 mm

clear-transparent


- lightly flexible consistency
- clear-transparent
- bonds with acrylate





Drufolen H 0.6 / 0.8 mm

natural coloured

- hard-flexible consistency
- burns without residue
- available in Ø 42 mm for manual adapters
- bonds with acrylate

Warm-up times  in minutes

Cooling times  in minutes and under pressure res. vacuum

	Drufosmart	Vacfomat U	Vacfomat 2000	
				
Biolon				
0.5 mm	1:00	0:55	1:00	1:00
0.75 mm	1:05	0:55	1:05	1:00
Drufolen H				
0.6 mm	1:30	1:20	1:00	1:20
0.8 mm	1:40	1:50	1:10	1:50

Barcode information for Drufomat-scan and Drufosmart-scan please find on page „Barcodes“

Biolon 0.5 mm

D-42300	clear-transparent	20 pieces
D-423001	clear-transparent	50 pieces

Biolon 0.75 mm

D-42303	clear-transparent	20 pieces
D-423031	clear-transparent	40 pieces

Drufolen H 0.6 mm

D-423641	natural coloured	40 pieces
----------	------------------	-----------

Drufolen H 0.8 mm

D-423681	natural coloured	40 pieces
----------	------------------	-----------

Production of temporary

Biolit D 2 mm

clear-transparent


- material can be sterilised
- predrying required
(4 hours at 80 °C per mm blank thickness)
- clear-transparent
- bonds with acrylate





Biolon 2 / 3 mm

clear-transparent

- lightly flexible consistency
- clear-transparent
- bonds with acrylate

Warm-up times  in minutes

Cooling times  in minutes and under pressure res. vacuum

	Drufosmart	Vacfomat U	Vacfomat 2000	
				
Biolit D 2 mm	2:15	2:15	2:05	2:15
Biolon 2 mm	1:45	1:50	1:30	1:50
3 mm	2:15	2:20	2:00	2:20

Barcode information for Drufomat-scan and Drufosmart-scan please find on page „Barcodes“

Biolit D 2 mm

D-4262 clear-transparent 10 pieces

Biolon 2 mm

D-42398 clear-transparent 15 pieces

D-423991 clear-transparent 50 pieces

Biolon 3 mm

D-4241 clear-transparent 10 pieces

D-42411 clear-transparent 30 pieces

Biolit D 1 / 2 mm

clear-transparent

- material can be sterilised
- predrying required
(4 hours at 80 °C per mm blank thickness)
- clear-transparent
- bonds with acrylate

Biolon 2 mm

clear-transparent


- lightly flexible consistency
- clear-transparent
- bonds with acrylate





Drufoplast (includes tray handle) **3 mm**

clear-transparent

- very break-resistant

Warm-up times  in minutes

Cooling times  in minutes and under pressure res. vacuum

	Drufosmart	Vacfomat U	Vacfomat 2000	
				
Biolit D				
1 mm	1:45	1:45	1:35	1:45
2 mm	2:15	2:15	2:05	2:15
Biolon				
2 mm	1:45	1:50	1:30	1:50
Drufoplast (incl. tray handle)				
3 mm	2:05	2:10	1:35	2:10

Barcode information for Drufomat-scan and Drufosmart-scan please find on page „Barcodes“

Biolit D 1 mm

D-4260 clear-transparent 10 pieces

Biolit D 2 mm

D-4262 clear-transparent 10 pieces

Biolon 2 mm

D-42398 clear-transparent 15 pieces

D-423991 clear-transparent 50 pieces

Drufoplast (includes tray handle) **3 mm**

D-4214 transparent 10 pieces

D-4215 transparent 100 pieces

Drufolen W 0.65 / 0.8 mm

blue or natural coloured


- soft-flexible consistency
- burns without residue
- does not bond with following blanks





UZF Underlining blanks 0.1 mm

red

- very thin, only thermoformable in combination with other blanks
- available in Ø 42 mm for manual adapters

Warm-up times  in minutes

Cooling times  in minutes and under pressure res. vacuum

	Drufosmart	Vacfomat U	Vacfomat 2000	
				
Drufolen W				
0.65 mm	1:15	1:05	0:50	2:00
0.8 mm	1:25	1:20	1:05	2:00
UZF				
0.1 mm	Data of the 2nd blank apply			

Barcode information for Drufomat-scan and Drufosmart-scan please find on page „Barcodes“

Drufolen W 0.65 mm

- D-423461B blue 40 pieces
- D-423461N natural coloured 40 pieces

Drufolen W 0.8 mm

- D-423481B blue 40 pieces
- D-423481N natural coloured 40 pieces

UZF Underlining blank 0.1 mm

- D-42470 red 100 pieces
- D-42470-42 red (Ø 42 mm) 100 pieces

Drufolen H 0.6 / 0.8 mm

natural coloured


- hard-flexible consistency
- burns without residue
- available in Ø 42 mm for manual adapters





Drufolen W 0.45 / 0.65 / 0.8 / 1 mm

blue or natural coloured

- soft-flexible consistency
- burns without residue
- available in Ø 42 mm for manual adapters

Warm-up times  in minutes

Cooling times  in minutes and under pressure res. vacuum

	Drufosmart	Vacfomat U	Vacfomat 2000	
				
Drufolen H				
0.6 mm	1:30	1:20	1:00	1:20
0.8 mm	1:40	1:50	1:10	1:50
Drufolen W				
0.45 mm	1:10	0:50	0:40	1:30
0.65 mm	1:15	1:05	0:50	2:00
0.8 mm	1:25	1:20	1:05	2:00
1 mm	1:40	1:40	1:20	2:30

Barcode information for Drufomat-scan and Drufosmart-scan please find on page „Barcodes“

Drufolen H 0.6 mm

D-423641 natural 40 pieces
D-42374N-42 natural (Ø 42 mm) 150 pieces

Drufolen H 0.8 mm

D-423681 natural 40 pieces

Drufolen W 0.45 mm

D-423441N natural 40 pieces
D-42384N-42 natural (Ø 42 mm) 150 pieces

Drufolen W 0.65 mm

D-423461N natural 40 pieces
D-42386N-42 natural (Ø 42 mm) 100 pieces
D-423461B blue 40 pieces

Drufolen W 0.8 mm

D-423481N natural 40 pieces
D-42388N-42 natural (Ø 42 mm) 100 pieces
D-423481B blue 40 pieces

Drufolen W 1 mm

D-423401N natural 30 pieces

BioBleach® hard 0.5 mm

clear-transparent

- for manufacturing of hard bleaching- / medical splints ending at the gingival margin
- soft-flexible, also suitable for situations with slight undercuts
- clear-transparent
- bonds with acrylate

BioBleach® soft 1 mm

transparent

- for manufacturing of soft remaining bleaching- / medical splints exceeding the gingival margin
- high form constancy, 95 Shore A





Drufosoft® 1 / 1.5 mm

transparent

- soft-remaining, 82 Shore A
- high power of retraction
- can be laminated

Warm-up times  in minutes

Cooling times  in minutes and under pressure res. vacuum

	Drufosmart	Vacformat U	Vacformat 2000	
				
BioBleach® hard 0.5 mm	1:00	0:55	1:00	1:00
BioBleach® soft 1 mm	1:05	1:05	0:50	3:00
Drufosoft® 1 mm	0:45	0:40	0:40	3:00
1.5 mm	1:00	0:55	0:50	3:00

Barcode information for Drufomat-scan and Drufosmart-scan please find on page „Barcodes“

BioBleach® hard 0.5 mm

D-4294 clear-transparent 20 pieces

BioBleach® soft 1 mm

D-4300 transparent 20 pieces

Drufosoft® 1 mm

D-4245 transparent 20 pieces

D-42451 transparent 50 pieces

Drufosoft® 1.5 mm

D-4252 transparent 20 pieces

D-42531 transparent 50 pieces

Biolit D 1 / 2 mm

clear-transparent

- material can be sterilised
- predrying required
(4 hours at 80 °C per mm blank thickness)
- clear-transparent
- bonds with acrylate

Biolon 0.75 / 1 / 1.5 / 2 / 3 mm

clear-transparent

- lightly flexible consistency
- clear-transparent
- bonds with acrylate

Drufosoft® pro 3 mm

transparent


- very hard-flexible, 95 Shore A
- high power of retraction
- can be laminated





Kombiplast 1.8 / 3 mm

transparent

- hard surface, soft rear side for
higher wearing comfort
- bonds with acrylate

Warm-up times  in minutes

Cooling times  in minutes and under pressure res. vacuum

	Drufosmart	Vacfomat U	Vacfomat 2000	
				
Biolit D				
1 mm	1:45	1:45	1:35	1:45
2 mm	2:15	2:15	2:05	2:15
Biolon				
0.75 mm	1:05	0:55	1:05	1:00
1 mm	1:10	1:00	1:10	1:10
1.5 mm	1:20	1:30	1:20	1:20
2 mm	1:45	1:50	1:30	1:50
3 mm	2:15	2:20	2:00	2:20
Drufosoft® pro				
3 mm	2:15	2:20	1:45	7:00
Kombiplast				
1.8 mm	1:30	--	1:05	2:30
3 mm	1:50	--	1:50	2:30

Barcode information for Drufomat-scan and Drufosmart-scan please find on page „Barcodes“

Biolit D 1 mm		
D-4260	clear-transparent	10 pieces
Biolit D 2 mm		
D-4262	clear-transparent	10 pieces
Biolon 0.75 mm		
D-42303	clear-transparent	20 pieces
D-423031	clear-transparent	40 pieces
Biolon 1 mm		
D-42396	clear-transparent	20 pieces
D-423971	clear-transparent	50 pieces
Biolon 1.5 mm		
D-42402	clear-transparent	20 pieces
D-424021	clear-transparent	50 pieces
Biolon 2 mm		
D-42398	clear-transparent	15 pieces
D-423991	clear-transparent	50 pieces
Biolon 3 mm		
D-4241	clear-transparent	10 pieces
D-42411	clear-transparent	30 pieces
Drufosoft® pro 3 mm		
D-4230	transparent	10 pieces
Kombiplast 1.8 mm		
D-4290	transparent	10 pieces
Kombiplast 3 mm		
D-4291	transparent	10 pieces

Biolon 2 mm

- used as rigid intermediate blank for improved force allocation
- lightly flexible consistency
- clear-transparent
- bonds with acrylate

Drufosoft®

- constantly soft, 82 Shore A
- high power of retraction
- can be laminated

Available transparent or coloured:

Drufosoft® 2 / 3 / 4 mm

Drufosoft® colour 3 mm

- available in many different colours (see colour map)

Drufosoft® bicolour 3 mm





- available in different colour combinations (see colour map)

Drufosoft® tricolour 3 mm

- available in different colour combinations (see colour map)

Warm-up times  in minutes

Cooling times  in minutes and under pressure res. vacuum

	Drufosmart	Vacfomat U	Vacfomat 2000	
				
Biolon 2 mm	1:45	1:50	1:30	1:50
Drufosoft® transparent 2 mm	1:05	1:10	1:00	3:30
3 mm	1:50	1:40	1:20	6:00
4 mm	2:10	2:15	1:30	7:00
Drufosoft® colour/ bicolour/ tricolour 3 mm	1:35-1:50	1:20-1:35	0:55-1:10	6:00

Barcode information for Drufomat-scan and Drufosmart-scan please find on page „Barcodes“

Biolon 2 mm

D-42398	clear-transparent	15 pieces
D-423991	clear-transparent	50 pieces

Drufosoft® 2 mm

D-4250	transparent	15 pieces
D-42511	transparent	50 pieces

Drufosoft® 3 mm

D-4248	transparent	10 pieces
D-42481	transparent	30 pieces

Drufosoft® 4 mm

D-42472	transparent	15 pieces
D-424901	transparent	75 pieces

Drufosoft® colour 3 mm

D-4248-2	neon red	10 pieces
D-4248-3	neon green	10 pieces
D-4248-4	neon yellow	10 pieces
D-4248-5	blue	10 pieces
D-4248-6	yellow	10 pieces
D-4248-7	black	10 pieces
D-4248-8	white	10 pieces
D-4248-9	pink	10 pieces
D-4248-17	red	10 pieces
D-4248-18	turquoise	10 pieces
D-4248-19	gold	10 pieces
D-4248-20	silver	10 pieces
D-4248-21	blue-opaque	10 pieces
D-4248-22	green	10 pieces
D-4248-23	lilac	10 pieces
D-4248-24	light-blue	10 pieces
D-4248-25	bordeaux	10 pieces
D-4248-26	orange	10 pieces

Drufosoft® bicolour 3 mm

D-4270-11	yellow / black / yellow	3 pieces
D-4270-12	white / blue / white	3 pieces
D-4270-13	red / blue / red	3 pieces

Drufosoft® tricolour 3 mm

D-4275-14	black / red / yellow	3 pieces
D-4275-15	blue / white / red	3 pieces
D-4275-16	red / green / blue	3 pieces

Drufosoft® colourmix 3 mm

D-4265	1 blank / colour	24 pieces
--------	------------------	-----------

Biolon 0.75 mm

clear-transparent

- lightly flexible consistency
- clear-transparent
- bonds with acrylate

Drufosoft®

- constantly soft, 82 Shore A
- high power of retraction
- can be laminated

Drufosoft® 1.5 / 2 / 3 / 4 mm

transparent

Drufosoft® colour 3 mm

- available in many different colours
(see colour map)

Drufosoft® bicolour 3 mm

- available in many different colours
(see colour map)

Drufosoft® tricolour 3 mm


- available in many different colours
(see colour map)





Drufosoft® pro 3 mm

transparent

- hard-flexible consistency, 95 Shore A
- high power of retraction
- can be laminated

Warm-up times  in minutes

Cooling times  in minutes and under pressure res. vacuum

	Drufosmart	Vacfomat U	Vacfomat 2000	
				
Biolon 0.75 mm	1:05	0:55	1:05	1:00
Drufosoft® transparent 1.5 mm	1:00	0:55	0:50	3:00
2 mm	1:05	1:10	1:00	3:30
3 mm	1:50	1:40	1:20	6:00
4 mm	2:10	2:15	1:30	7:00
5 mm	2:45	--	2:00	8:30
Drufosoft® colour/ bicolour/ tricolour 3 mm	1:35-1:50	1:20-1:35	0:55-1:10	6:00
Drufosoft® colour 4 mm	2:20-2:35	2:25-2:40	1:45-2:00	7:10
Drufosoft® pro 3 mm	2:15	2:20	1:45	7:00

Barcode information for Drufomat-scan and Drufosmart-scan please find on page „Barcodes“

Biolon 0.75 mm

D-42303	clear-transparent	20 pieces
D-423031	clear-transparent	40 pieces

Drufosoft® 1.5 mm

D-4252	transparent	20 pieces
D-42531	transparent	50 pieces

Drufosoft® 2 mm

D-4250	transparent	15 pieces
D-42511	transparent	50 pieces

Drufosoft® 3 mm

D-4248	transparent	10 pieces
D-42481	transparent	30 pieces

Drufosoft® 4 mm

D-42427	transparent	15 pieces
D-424901	transparent	75 pieces

Drufosoft® 5 mm

D-4225	transparent	15 pieces
--------	-------------	-----------

Drufosoft® colour 3 mm

D-4248-2	neon red	10 pieces
D-4248-3	neon green	10 pieces
D-4248-4	neon yellow	10 pieces
D-4248-5	blue	10 pieces
D-4248-6	yellow	10 pieces
D-4248-7	black	10 pieces
D-4248-8	white	10 pieces
D-4248-9	pink	10 pieces
D-4248-17	red	10 pieces
D-4248-18	turquoise	10 pieces
D-4248-19	gold	10 pieces
D-4248-20	silver	10 pieces
D-4248-21	blue-opaque	10 pieces
D-4248-22	green	10 pieces
D-4248-23	lilac	10 pieces
D-4248-24	light-blue	10 pieces
D-4248-25	bordeaux	10 pieces
D-4248-26	orange	10 pieces

Drufosoft® colour 4 mm

For ordering this materials, please replace the item number D-4248 of Drufosoft® colour 3 mm Drufosoft® bicolour 3 mm by the item number D-4247

Drufosoft® bicolour 3 mm

D-4270-11	yellow / black / yellow	3 pieces
D-4270-12	white / blue /white	3 pieces
D-4270-13	red / blue / red	3 pieces

Drufosoft® tricolour 3 mm

D-4275-14	black / red / yellow	3 pieces
D-4275-15	blue / white / red	3 pieces
D-4275-16	red / green / blue	3 pieces

Drufosoft® colourmix 3 mm

D-4265	1 blank / colour	24 Pieces
--------	------------------	-----------

Drufosoft® pro 3 mm

D-4230	transparent	10 pieces
--------	-------------	-----------



BioBleach hard 0.5



BioBleach soft 1.0



Biolit D 1.0



Biolit D 2.0



Biolon 0.5



Biolon 0.75



Biolon 1.0



Biolon 1.5



Biolon 2.0



Biolon 3.0



Drufolen H 0.6



Drufolen H 0.8



Drufolen W 0.8



Drufolen W 0.45



Drufolen W 0.65



Drufolen W 1.0



Drufoplast 2.0



Drufoplast rosa 3.0



Drufoplast transparent 3.0



Drufosoft 1.0



Drufosoft 1.5



Drufosoft 2.0



Drufosoft 3.0



Drufosoft 4.0



Drufosoft 5.0



Drufosoft pro 3.0



Kombiplast 1.8



Kombiplast 3.0

	transparent
	neon red
	neon green
	neon yellow
	blue
	yellow
	black
	white
	pink
	red
	orange
	turquoise
	gold
	silver
	blue-opaque
	green
	lilac
	light blue
	bordeaux
	yellow / black / yellow*
	white / blue / white*
	red / blue / red*
	black / red / gold*
	blue / white / red*
	red / green / blue*

* available in 120 mm standard size