Modo de empleo FKO-Split Fixator

Estimado cliente

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Instructions for use FKO-Split Fixator

Dear Customer

Thank you for choosing this high-quality product from Dentaforum.

In order to achieve the best results with this product, it is important to carefully study and follow these instructions for use.

The written instructions cannot cover all possible situations that can occur during use. For this reason, our hotline is available to answer any other questions or concerns that may arise.

Due to constant developments, we recommend, even when using the same products regularly, that you study the enclosed current instructions for use or refer to our website at www.dentaforum.de.

1. Manufacturer

Dentaforum GmbH & Co. KG | Turnstr. 31 | 75228 Ispringen | Germany

2. With reference to our quality standards

Dentaforum ensures a faultless quality of the products manufactured by us. These recommendations are based upon our own experiences. The user himself is responsible for the processing of the products. Responsibility for failures cannot be taken, as we have no influence on the processing on site.

3. Description

The FKO-Split Post Fixator (Fig. 1) consists of:

- upper and lower fixation arms for the plaster models
- bite lock with fixation screw
- two round, parallel telescopes

The FKO-Split Post Fixator is designed for precise occlusal positioning and holding and has a vertically adjustable upper fixation arm. The two round telescopes prevent rotation of the horizontal axis. The conical fixation arms with retention channels allow for removal and exact replacement of the plaster models. The central screws enable the models to be held securely in position. The movable bite lock secures the vertical occlusal position given by the bite registration.

4. How to use the FKO-Split Post Fixator

1. The central screws on the fixation arms must be screwed right the way in.
2. The plaster models should be fixed with the aid of a bite registration. Apply articulating plaster to the base of the upper and lower models and place midway between the ends of the central screws.
3. Complete the model articulation in the FKO-Split Post Fixator and ensure that the plaster on the upper model is flush with the upper fixation arm (Fig. 2).
4. Before it finally sets, remove any excess plaster from the upper surface of the fixator arm and from the screw slits.
5. After having fixed with plaster, move the bite lock all the way up to the upper fixator arm and screw in tightly (Fig. 3).

5. Removing the models

1. Separate the upper fixator arm from the lower.
2. Unscrew and remove the central screws using a screwdriver.
3. Remove the models with their plaster bases from the fixator arms, taking care not to tilt to one side or the other.
4. When replacing the models, check that the fixator arms are clean (free of plaster and wax residues). If possible, clean the fixator arms and model base using a steam cleaner.
5. Replace the models onto the fixator arms carefully without tilting.
6. Using a screwdriver, turn the central screws all the way back inside and subsequently secure the models.

6. Tips

- Seal the slits in the central screws to avoid plaster from entering.
- Spray the FKO-Split Post Fixator with a thin layer of silicon spray to allow easy removal of the models later.
- Any excess plaster on the sides of the upper and lower model can be easily removed using the plaster trimmer.
- Placing the plaster models and their bases into the archives saves time later. Repeating articulation is unnecessary should FKO appliance repairs be required. Simply slide the old models back onto the FKO-Split Post Fixator arms and secure with the central screws.

Caution: never strike the FKO-Split Post Fixator with a hammer or similar instrument in order to remove the models and plaster bases. This will distort the FKO-Split Post Fixator and damage its function.

Note: For manufacturing of horizontal split activators, we recommend the additional kit for the FKO-Split Post Fixator.

7. Availability

FKO-Split Post Fixator	1 piece	REF 072-004-00
Additional kit for the FKO-Split Post Fixator	1 piece	REF 072-006-00

Date of information: 02/12
Subject to modifications

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