Surfactant - Debubblizer

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Debubblizer - bubble free models and casts

- Reduces surface tension on wax and impressions.
- Works on Polyether (i.e. Impregum™) and Alginate.
- Fine mist doesn't require drying time.
- Allows gypsum and investment to flow smoothly into all areas.
- Non-contaminating and alcohol free.

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Surfactant Debubblizer - bubble free models and casts

Disinfect the impressions as usual. After drying, spray on a thin coat of Surfactant on polyether and silicon impressions, or wax-ups. There is no need to use compressed air to dry the surface after coating, nor do you need to rinse with water! In case you see excess material creating puddles, simply „shake out“ the impression. The coated impression is now slowly poured with plaster as usual, or wax structures are invested as usual.

The plaster or investment material is now crowding out the surfactant coat and you will get a bubble free result.

For pouring alginate impressions, we recommend to rinse under running water after disinfecting, to remove the algin-acid layer. Then carefully remove any puddles of water by using compressed air. Now proceed as described above.

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