

Itsoclear

HIGH QUALITY CRYSTAL CLEAR CLASP

Itsoclear provides you with the ability to produce and repair a clasp in under 4 minutes with a cosmetic appeal unlike any other clasp.

Itsoclear's superior difference stands out from all the rest

- Saves bench time producing a higher quality dental esthetic clasp that your customers will appreciate while increasing your profit.
- Crystal clear, will not lose its clarity
- Non-abrasive
- Easily adjustable
- No injection machines or hand-held injectors required
- Perfect for new partial dentures as well as repairs and replacing metal clasp



4 easy steps to create your Itsoclear clasp in under 4 minutes

- 1A)** Warm clasp by lightly waving it over flame. (WARNING: Do not overheat the clasp)
 - 1B)** Lightly pass 3 prong plier over flame. (WARNING: Overheating plier will cause indentation in clasp)
 - 1C)** Adapt accordingly around tooth by rewarming the 3 prong plier as needed.
 - 2A)** After clasp is formed around tooth begin to create retention.
 - 2B)** Form retention by bending a loop at opposite end of clasp to hold in acrylic.
 - 2C)** Once retention is formed the clasp is ready to be set in place on frame.
 - 3)** Secure clasp on frame by using glue and accelerator.
 - 4)** Finally, place acrylic under retentive area and above the clasp so that the clasp will remain in position.
- Your partial denture is now ready to be processed as normal.

REPAIRS or REPLACEMENTS: Cut groove where clasp is to be formed and cut a deep groove in tentative area. Process as normal.

Clear Universal Clasps, 6/Pkg. *Recommended use on pre-molar/anterior teeth*

Clear Molar Clasps, 6/Pkg. *Recommended use when stronger reinforcement is needed*

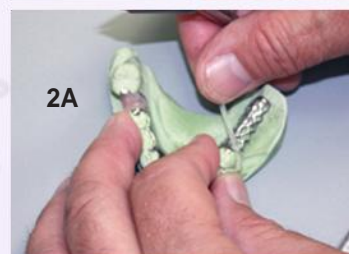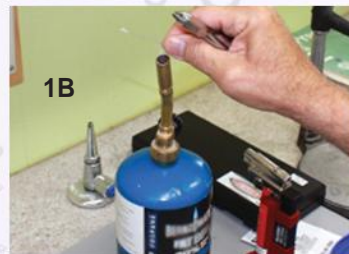

**Wrought Wire
Clasp**



**Wrought Wire
vs Itsoclear**



Itsoclear

itsoclear

Visit us at www.whwplastics.com

whw
Behind every smile