
## Ref. 100276

|          |                                   |   |
|----------|-----------------------------------|---|
| ES ..... | PULIDORA VELOCIDAD VARIABLE ..... | 4 |
| EN ..... | POLISHER VARIABLE SPEED .....     | 6 |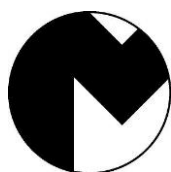

**MESTRA®**

**TALLERES MESTRAITUA S.L.**

Txori-Erri Etorbidea, 60

Tel. +34 944530388 - Fax +34 944711725

mestra@mestra.es - www.mestra.es

**48150 SONDIKA - BILBAO - ESPAÑA**

Rev. 02/09/2020

- |   |  |
|---|--|
| A | Portacepillos / <b>Brush holder</b>                          |
| B | Regulador velocidad / <b>Speed regulator</b>                 |
| C | Disco de tela / <b>Cloth wheel</b>                           |
| D | Tornillos sujeción portacepillos / <b>Brush holder bolts</b> |
| E | Tapón escobillas / <b>Brush cover</b>                        |
| F | Interruptor general / <b>Main switch</b>                     |
| G | Patas de goma / <b>Rubber legs</b>                           |

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The Ref. 100276 is a simple polishing lathe especially designed to polish and grind small items in the field of dental mechanics.

## INSTALLATION

1. Carefully unpack the unit. Check that it has not sustained damage during shipment.

It is supplied with the following accessories:

- 1 right-hand thread brush holder.
  - 1 left-hand thread brush holder.
  - 1 cloth wheel Ø100.
  - 1 pair of brushes.
  - 1 Allen wrench.
2. Place the unit on a flat and firm surface remote from any source of heat or vibration.
  3. Connect the unit to a 230 V, 50/60 Hz outlet with earth.

## FITTING THE BRUSH TO THE BRUSH HOLDER

1. Insert the brush into the brush-holder, ensuring that the brush-holder shaft is perpendicular to the plane of the brush.
2. Screw the brush onto the brush-holder as tightly as possible by hand.
3. Visually check that the brush-holder shaft is perpendicular to the plane of the brush by spinning the assembly in your hand.
4. Start up the unit: the spin of the motor will reveal whether the wheel or brush is off line. If there is more than 2 mm of slip, dismount and re-fit the wheel-holder or brush-holder.
5. Remember that the friction torque produced during work after a brush is fitted to the brush-holder will keep it tight. This effect can be favoured by a suitable workpiece attack angle.

**Note.** Max. disc diameter: 125 mm.

## CONDITIONS OF USE

For best results use the unit as follows:

- Connect the unit to a 230 V, 50/60 Hz power source.
- Turn on the main switch (G).
- Select the desired spin speed (500-8000 rpm) turning selector (B).
- Before polishing or grinding a workpiece, check the motor spin direction, as the initial workpiece attack angle depends on this. Incorrect attack angles can be extremely dangerous.
- The following protective equipment is recommended for polishing or grinding:
  - safety goggles;
  - safety gauntlets;
  - face-mask;
  - proper exhausters;
  - a protective screen suited to the characteristics and shape of the unit. TALLERES MESTRAITUA, S.L. recommends Ref. 080200 / 080210 screens, which are especially designed for use with the polisher.
  - Proper clothes. No baggy clothes, ties, necklaces, rings, bracelets or elements that could be hooked by the machine.
  - Be careful with long hair.
  - No objects similar to ropes or chains that could be hooked by the machine.
- After using the unit, clean the inside of the protective screen.

## MAINTENANCE AND CLEANING

To ensure a long, trouble-free useful life for your unit, proceed as follows:

- To clean the motor and base use a cloth. Never use alcohol, solvents or other inflammable liquids.
- Never immerse the unit in any liquid.
- If you notice unusual noises or vibrations, send the unit to your technical assistance service.
- This unit is designed to work correctly on the four rubber legs (C).
- The motor has two brushes that will wear out with the use. To replace them, remove the plugs (D) using a screwdriver.

## PRECAUTIONS

- Before connecting the unit to the mains, check that the power supply is 230 V, 50/60 Hz.
- Discard any cracked or damaged brush or grinding wheel: if these elements break during operation, they could cause serious accidents.
- Before starting work, ensure that the brush-holder/wheel holder is firmly attached to the motor shaft.
- When handling workpieces, use safety goggles, gauntlets, a face mask and a suitable safety screen. We recommend MESTRA Ref. 080200 / 080210 screens.
- Do not allow children or unskilled personnel to handle the unit.
- If the motor shaft locks during normal operation, disconnect the unit from the mains before attempting to release it by turning it manually.
- Never polish, brush or grind materials which may give off particles which are toxic if inhaled.
- Before starting to polish or grind a workpiece, check the motor spin direction: the attack angle depends on this direction.
- Never use the unit close to solvents, alcohol or other inflammable materials. Take all possible precautions when grinding materials which may give off sparks.
- Never use the unit without proper exhausters.
- Keep the working area clean and tidy in order to prevent accidents.
- Wear proper working cloth. Loose cloth, hanging or long hair could be caught by the machine.
- Make sure that you switch the power Off before plugging the unit, in order to avoid sudden start of the motor.
- Disconnect the unit from power supply before changing the wheel.

## TECHNICAL CHARACTERISTICS

|                              |                 |
|------------------------------|-----------------|
| Power supply:                | 230 V, 50/60 Hz |
| Power:                       | 1300 W          |
| Speed:                       | 500-8000 rpm    |
| Height:                      | 16,5 cm         |
| Height of shaft centre line: | 12 cm           |
| Width with brush holders:    | 36 cm           |
| Weight:                      | 6 kg            |