HS Cross-Liquid

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Instructions for use

Application:

Echant for cross-linked teeth and acrylics.

Application technique:

Apply a thin coat of HS Cross-Liquid to the mechanically roughened teeth. Ensure that there are no pools of liquid. Allow the prepared teeth to dry in air for 2 - 3 min. Then apply the acrylic.

Storage:

Store at 10 - 25 °C.

Do not use after the expiry date.

Supplied as:

50 ml liquid

Caution:

Avoid skin contact with the liquid and inhalation of fumes. Opened containers: Do not use after the expiry date.

Disposal:

The liquid should be disposed of as hazardous waste.

Side-effects:

With proper use of this medical device, unwanted side-effects are extremely rare. Reactions of the immune system (allergies) or local discomfort, however, cannot be ruled out completely. Should you learn about unwanted side-effects – even if it is doubtful that the side-effect has been caused by our product – please kindly contact us.

Contra-indications / interactions:

If a patient has known allergies against or hypersensitivities towards a component of this product, we recommend not to use it or to do so only under strict medical supervision. In such cases, we will supply the composition of our medical device upon request. The dentist should consider known interactions and crossreactions of the product with other materials already in the patient's mouth before using the product.

Note:

Please supply the dentist with the above information, if this medical device is used to produce a special model. Please also consider the safety data sheets.

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