

VITA VITABLOCS®

Processing recommendation for CAD/CAM systems



VITA shade determination

VITA shade communication

VITA shade reproduction

VITA shade control

VITA – perfect match.

VITA

Machining mode: Grinding – Block & Disc

- Information and tips
- Tools
- Machining strategy
- Parameters

Information

The information presented here, are intended as a recommendation.

Depending on the available CNC machines, CAM software, tools, etc. the information have to be adapted to your own production situation.

As a result, different results may obtained.

The development of the strategies and parameter was done with following system:

- imes-icore CORiTEC350i
- CAM Software: Hyperdent 8.1.3

According to this recommendation, a fully anatomical posterior tooth crown (tooth 26) can be finished in 00:28:30 min, with a good surface and fit.

We recommend Tools from:

FRANKEN GmbH & Co. KG, Fabrik für Präzisionswerkzeuge

www.franken-dental.com

Tips for VITA VITABLOCS®

Avoid vertically or fast plunge movements. It is important that the tool always plunge slow and soft into the material.

- Plunge into the material with a ramp (5 degree) or helically and use a reduced plunge feed (feed Z)
- We recommend to grind VITA VITABLOCS wet
- The diameter of the restoration holding pin should be 3,0 - 3,5 mm

Strategy

- A two side machining and 3+2 strategies are sufficient in most cases.
- In order to maintain a good fit, even by restoration with undercuts, the last finishing of the cavity should be done with a 5 axis strategy.
- In order to maintain a good occlusal fit, the complete occlusal side should be finished with max. a $\varnothing 1.2\text{mm}$ tool (or less). In that way, a special finishing of the fissures isn't necessary.
- If a smaller tool is used after a bigger one, it can be necessary to use a roughing strategy to remove remaining material.
- Tool life and process reliability are increased this way.
- To process cavities or pockets, the tool should be tilted 4-7 degrees (5 axis strategy). This will decrease the wear of the tool tip.
- When using grinding tools, the whole grinding body should be used.



Recommended Tools

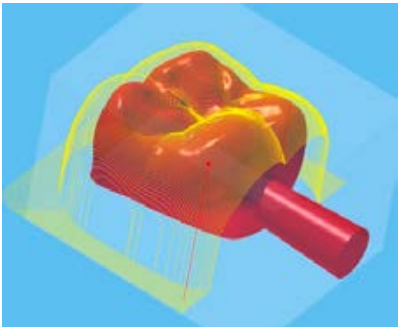
| Diameter | Grain size | Description | Manufacturer | Order-Code |
|----------------------------|------------|---------------------------------|---------------|----------------------------|
| $\varnothing 3 \text{ mm}$ | D126 | Diamond ball nose grinding burr | Emuge-Franken | 1716.300613 (6mm shaft) |
| $\varnothing 2 \text{ mm}$ | D126 | Diamond ball nose grinding burr | Emuge-Franken | 1716.200611 (6mm shaft) |
| $\varnothing 1 \text{ mm}$ | D76 | Diamond ball nose grinding burr | Emuge-Franken | 1716.100609 (6mm shaft) |

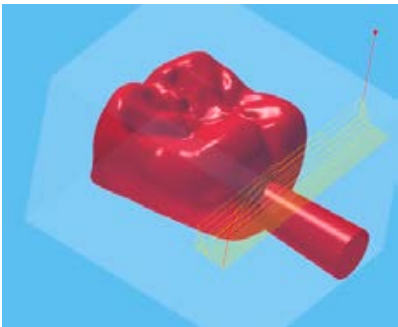
Tool Life

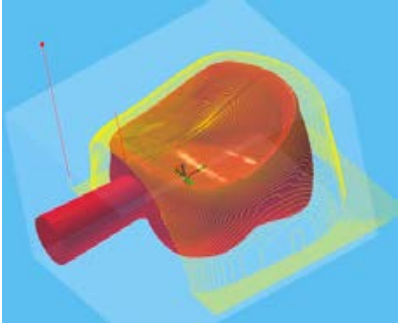
| Tool | Units | Restoration |
|--|-------|---------------------------------|
| Ø 3 mm Diamond ball nose grinding burr | 137 | Fully anatomical crown tooth 26 |
| Ø 2 mm Diamond ball nose grinding burr | 146 | Fully anatomical crown tooth 26 |
| Ø 1 mm Diamond ball nose grinding burr | 70 | Fully anatomical crown tooth 26 |

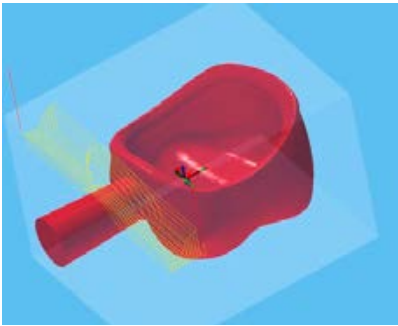
Order of machining


| Step | Machining side | Machining | Tool |
|------|----------------------------|--------------------|-------|
| 1 | Occlusal side | Roughing 1 | Ø 3mm |
| 2 | Occlusal side | Roughing 2 | Ø 3mm |
| 3 | Cavity side | Roughing 1 | Ø 3mm |
| 4 | Cavity side | Roughing 2 | Ø 3mm |
| 5 | Preparation margin inside | Pre-Finishing | Ø 2mm |
| 6 | Preparation margin outside | Pre-Finishing | Ø 2mm |
| 7 | Cavity, inside | Roughing | Ø 2mm |
| 8 | Cavity, outside | Finishing | Ø 2mm |
| 9 | Occlusal side | Pre-Finishing | Ø 2mm |
| 10 | Preparation margin inside | Finishing | Ø 1mm |
| 11 | Preparation margin outside | Finishing | Ø 1mm |
| 12 | Cavity, inside | Finishing | Ø 1mm |
| 13 | Cavity, inside | Remaining material | Ø 1mm |
| 14 | Occlusal side, fissures | Finishing | Ø 1mm |

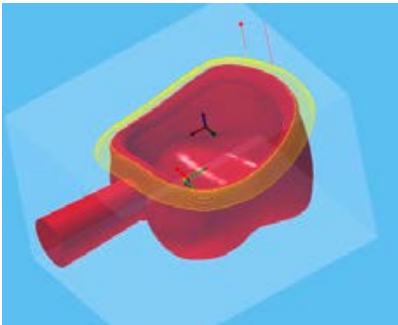
| Step 1 | Occlusal side - roughing 1 | | | 3 axis | |
|---|----------------------------|-------|-----------|--------|--------|
|  | Tool | Ø 3mm | | notes: | |
| | Tolerance | 0,01 | 0,01 | | |
| | Spindel speed | [n] | 45000 | | rpm |
| | Feed speed XY | [Vf] | 1800 | | mm/min |
| | Feed speed Z | [Vf] | 500 | | mm/min |
| | Width of cut XY | [ae] | 0,12 | | mm |
| | Depth of cut Z | [ap] | Full Tool | | mm |
| | Oversize | | 0,15 | | mm |


| Step 2 | Occlusal side - roughing 2 | | | 3 axis | |
|--|----------------------------|-------|------------------|--------|--------|
|  | Tool | Ø 3mm | | notes: | |
| | Tolerance | 0,01 | 0,01 | | |
| | Spindel speed | [n] | 45000 | | rpm |
| | Feed speed XY | [Vf] | 1500 | | mm/min |
| | Feed speed Z | [Vf] | 500 | | mm/min |
| | Width of cut XY | [ae] | 0,1 | | mm |
| | Depth of cut Z | [ap] | 1/3 Block height | | mm |
| | Oversize | | 0,15 | | mm |

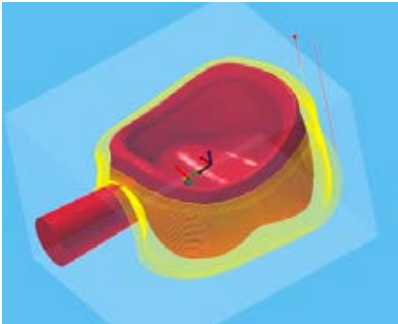
| Step 3 | Cavity side - roughing 1 | | | 3 axis | |
|---|--------------------------|-------|-----------|--------|--------|
|  | Tool | Ø 3mm | | notes: | |
| | Tolerance | 0,01 | 0,01 | | |
| | Spindel speed | [n] | 45000 | | rpm |
| | Feed speed XY | [Vf] | 1800 | | mm/min |
| | Feed speed Z | [Vf] | 500 | | mm/min |
| | Width of cut XY | [ae] | 0,12 | | mm |
| | Depth of cut Z | [ap] | Full Tool | | mm |
| | Oversize | | 0,15 | | mm |


| Step 4 | Cavity side - roughing 2 | | | 3 axis | |
|---|--------------------------|-------|------------------|--------|--------|
|  | Tool | Ø 3mm | | notes: | |
| | Tolerance | 0,01 | 0,01 | | |
| | Spindel speed | [n] | 45000 | | rpm |
| | Feed speed XY | [Vf] | 1500 | | mm/min |
| | Feed speed Z | [Vf] | 500 | | mm/min |
| | Width of cut XY | [ae] | 0,1 | | mm |
| | Depth of cut Z | [ap] | 1/3 Block height | | mm |
| | Oversize | | 0,07 | | mm |


| Step 5 | Preperation margin inside - pre-finishing | | | 3+2 axis | |
|---|---|-------|-------|----------|--------|
|  | Tool | Ø 2mm | | notes: | |
| | Tolerance | 0,01 | 0,01 | | |
| | Spindel speed | [n] | 45000 | | rpm |
| | Feed speed XY | [Vf] | 1300 | | mm/min |
| | Feed speed Z | [Vf] | 1000 | | mm/min |
| | Width of cut XY | [ae] | 0,1 | | mm |
| | Depth of cut Z | [ap] | - | | mm |
| | Oversize | | 0,07 | | mm |

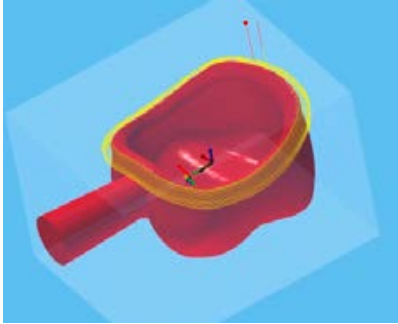
| Step 6 | Preperation margin outside - pre-finishing | | | 3+2 axis | |
|--|--|-------|-------|----------|--------|
|  | Tool | Ø 2mm | | notes: | |
| | Tolerance | 0,01 | 0,01 | | |
| | Spindel speed | [n] | 45000 | | rpm |
| | Feed speed XY | [Vf] | 1300 | | mm/min |
| | Feed speed Z | [Vf] | 1000 | | mm/min |
| | Width of cut XY | [ae] | 0,1 | | mm |
| | Depth of cut Z | [ap] | - | | mm |
| | Oversize | | 0.07 | | mm |


| Step 7 | Cavity, inside - roughing | | | 3+2 axis | |
|---|---------------------------|-------|-------|----------|--------|
|  | Tool | Ø 2mm | | notes: | |
| | Tolerance | 0,01 | 0,01 | | |
| | Spindel speed | [n] | 45000 | | rpm |
| | Feed speed XY | [Vf] | 1200 | | mm/min |
| | Feed speed Z | [Vf] | 500 | | mm/min |
| | Width of cut XY | [ae] | 0,1 | | mm |
| | Depth of cut Z | [ap] | - | | mm |
| | Oversize | | 0,0 | | mm |


| Step 8 | Cavity, outside - finishing | | | 3+2 axis | |
|---|-----------------------------|-------|-------|----------|--------|
|  | Tool | Ø 2mm | | notes: | |
| | Tolerance | 0,01 | 0,01 | | |
| | Spindel speed | [n] | 45000 | | rpm |
| | Feed speed XY | [Vf] | 1300 | | mm/min |
| | Feed speed Z | [Vf] | 500 | | mm/min |
| | Width of cut XY | [ae] | 0,1 | | mm |
| | Depth of cut Z | [ap] | - | | mm |
| | Oversize | | 0,0 | | mm |


| Step 9 | Occlusal side - pre-finishing | | | 3+2 axis | |
|---|-------------------------------|-------|-------|----------|--------|
|  | Tool | Ø 2mm | | notes: | |
| | Tolerance | 0,01 | 0,01 | | |
| | Spindel speed | [n] | 45000 | | rpm |
| | Feed speed XY | [Vf] | 1300 | | mm/min |
| | Feed speed Z | [Vf] | 500 | | mm/min |
| | Width of cut XY | [ae] | 0,12 | | mm |
| | Depth of cut Z | [ap] | - | | mm |
| | Oversize | | 0,02 | | mm |

| Step 10 | Preparation margin inside - finishing | | | 5 axis | |
|--|---------------------------------------|-------|-------|--------|--------|
|  | Tool | Ø 1mm | | notes: | |
| | Tolerance | 0,01 | 0,01 | | |
| | Spindel speed | [n] | 45000 | | rpm |
| | Feed speed XY | [Vf] | 1000 | | mm/min |
| | Feed speed Z | [Vf] | 1000 | | mm/min |
| | Width of cut XY | [ae] | 0,1 | | mm |
| | Depth of cut Z | [ap] | - | | mm |
| | Oversize | | 0,0 | | mm |

| Step 11 | Preparation margin outside - finishing | | | 5 axis | |
|---|--|-------|-------|--------|--------|
|  | Tool | Ø 1mm | | notes: | |
| | Tolerance | 0,01 | 0,01 | | |
| | Spindel speed | [n] | 45000 | | rpm |
| | Feed speed XY | [Vf] | 1000 | | mm/min |
| | Feed speed Z | [Vf] | 1000 | | mm/min |
| | Width of cut XY | [ae] | 0,1 | | mm |
| | Depth of cut Z | [ap] | - | | mm |
| | Oversize | | 0,0 | | mm |

| Step 12 | Cavity inside - finishing | | | 5 axis | |
|---|---------------------------|-------|-------|--------|--------|
|  | Tool | Ø 1mm | | notes: | |
| | Tolerance | 0,01 | 0,01 | | |
| | Spindel speed | [n] | 45000 | | rpm |
| | Feed speed XY | [Vf] | 1200 | | mm/min |
| | Feed speed Z | [Vf] | 1000 | | mm/min |
| | Width of cut XY | [ae] | 0,2 | | mm |
| | Depth of cut Z | [ap] | - | | mm |
| | Oversize | | 0,0 | | mm |

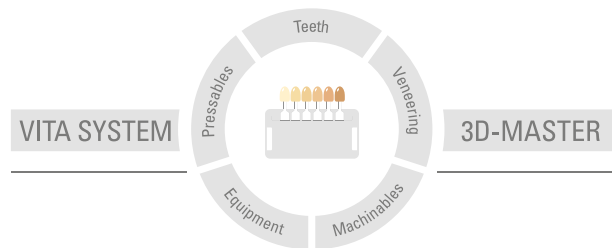
| Step 13 | Cavity - remaining material | | | 5 axis | |
|---|-----------------------------|-------|-------|--------|--------|
|  | Tool | Ø 1mm | | notes: | |
| | Tolerance | 0,01 | 0,01 | | |
| | Spindel speed | [n] | 45000 | | rpm |
| | Feed speed XY | [Vf] | 500 | | mm/min |
| | Feed speed Z | [Vf] | 250 | | mm/min |
| | Width of cut XY | [ae] | 0,1 | | mm |
| | Depth of cut Z | [ap] | 0,05 | | mm |
| | Oversize | | 0,0 | | mm |

| Step 14 | Occlusalside, fissures - finishing | | | 3+2 axis | |
|--|------------------------------------|-------|-------|----------|--------|
|  | Tool | Ø 1mm | | notes: | |
| | Tolerance | 0,01 | 0,01 | | |
| | Spindel speed | [n] | 45000 | | rpm |
| | Feed speed XY | [Vf] | 1500 | | mm/min |
| | Feed speed Z | [Vf] | 1000 | | mm/min |
| | Width of cut XY | [ae] | 0,1 | | mm |
| | Depth of cut Z | [ap] | - | | mm |
| | Oversize | | 0,0 | | mm |

Formulas for cutting data calculation

| Expression used in Text | Term | Symbol | Formula |
|-------------------------------|-----------------------|-------------|---------------------------------|
| Feed speed XY Feed speed Z | Feed speed | Vf [mm/min] | $Vf = fz * z * n$ |
| Spindle speed | Spindle speed | n [rpm] | $n = \frac{Vc * 1000}{\pi * d}$ |
| Width of cut XY | Width of cut | ae [mm] | |
| Depth of cut Z | Depth of cut | ap [mm] | |
| | Feed per cutting edge | fz [mm] | $fz = \frac{Vf}{n * z}$ |
| | Cutting speed | Vc [m/min] | $Vc = \frac{\pi * d * n}{1000}$ |

More information about **VITA CAD/CAM MATERIALS** is available at: www.vita-zahnfabrik.com/cadcam



Please note: Our products must be used in accordance with the instructions for use. We accept no liability for any damage resulting from incorrect handling or usage. The user is furthermore obliged to check the product before use with regard to its suitability for the intended area of applications. We cannot accept any liability if the product is used in conjunction with materials and equipment from other manufacturers that are not compatible or not authorized for use with our product and this results in damage. The VITA Modulbox is not necessarily a component of the product. Date of issue of this information:

After the publication of this information for use any previous versions become obsolete. The current version can be found at www.vita-zahnfabrik.com

VITA Zahnfabrik has been certified and the following products bear the CE mark:
CE 1024

VITA

VITA Zahnfabrik H. Rauter GmbH & Co.KG
Spitalgasse 3 · D-79713 Bad Säckingen · Germany
Tel. +49(0)7761/562-0 · Fax +49(0)7761/562-299
Hotline: Tel. +49(0)7761/562-222 · Fax +49(0)7761/562-446
www.vita-zahnfabrik.com · info@vita-zahnfabrik.com
 facebook.com/vita.zahnfabrik